



These contacts may be used in conjunction with Gentner's BC-10B for quick and easy recharging. Simply place the Digital-6 Plus receiver in the BC-10B charger. When not in use and it will automatically recharge for the next usage.

Important Recharging Information:

Fully depleted Ni-Cad batteries require approximately 18 hours to recharge. Once charged, Ni-Cad batteries will power the Digital-6 Plus for approximately 30 hours. Alkaline batteries will power the Digital-6 Plus for approximately 40 hours.

WARNING!

Use ONLY NI-Card rechargeable batteries in the Digital-6 Plus when using the BC-10B battery charger. Alkaline batteries will damage the BC-10B. Also, do not exceed 60 hours of continuous recharging or you may damage the batteries and/or the receiver.